



## 2000-206 Pads Kit, Liner Closed Cell Foam P/N 03521072



### WARNINGS:

Use only if package is unopened and undamaged. If packaging or any internal components appear damaged, contact the shipper for replacement or repair.

Only trained and experienced personnel should use this product. Before using any MAXAIR System or MAXAIR Systems component, read and understand the instructions. Pay special attention to WARNING information. Become familiar with the systems' NIOSH approved configurations prior to use.

Failure to follow user instructions may be hazardous to the user's health.

The institution or facility using this product in any application is responsible for determining the appropriateness of this equipment relative to regulatory requirements. Bio-Medical Devices Intl, Inc as a manufacturer, does not recommend the appropriate systems for a particular institution or facility.

Use only MAXAIR Systems/ NIOSH approved compatible components.

Prior to use, inspect each system component for damage. DO NOT use any system or component if any component is damaged.

If you need more information, contact your BMDI sales representative or call BMDI customer service at 1-800-443-3842.

### Intended Use

The MAXAIR Systems Liner Closed Cell Foam Pads Kit (2000-206) is intended to provide replacement comfort pads for the MAXAIR 2071-01 Liner (P/N 03531021), which is a component of the MAXAIR CAPR SYSTEM's 2070-03 Helmet (P/N 03531001).

The comfort pads are to provide a comfortable fit to the top of the wearer's head while using the MAXAIR CAPR System.

If any of the comfort pads initially installed on the liner become damaged or contaminated and therefore unfit for safe and effective use, they should be replaced.


Replacing the comfort pads simply requires peeling the unfit pad off and placing the new, replacement pad in the same place and position.

### Description

The comfort pads in the MAXAIR 2000-206 kit are made of closed cell foam. They will last indefinitely with normal use and handling.

To de-con the exposed portions of the MAXAIR Helmet System, including the Liner and the closed cell foam pads, simply wipe with an alcohol, bleach, or quaternary ammonium wipe and allow to dry.

Each Kit consists of the following:

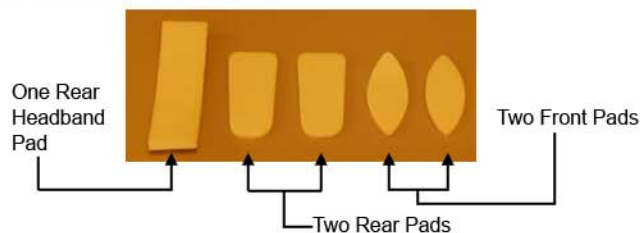

### CAUTIONS:

- Always inspect the components of the MAXAIR system before use.
- Do not use if any components appear damaged or unfit for use in the intended application.

### Changing Pads Instructions:



Grasp the comfort pad to be removed by one edge and slowly, gently, and firmly pull up and off the Liner.



Place the replacement pad in the same position as the one just removed and gently press it against the liner.



Press all along the surface of the replacement pad against the liner to ensure the pad is fully adhered to the liner.



Grasp one end edge of the rear headband comfort pad and pull the pad up and off the rear headband.

**NOTE:** Front and rear pads, right and left side, are attached similar as A.-C.



With the replacement rear headband comfort pad loop-attachment-side facing the rear headband, position the replacement pad directly above and centered on the hook attachment area on the rear headband.



Press the entire surface of the replacement pad firmly against the rear headband. Ensure all edges of the replacement pad loop are attached to the rear headband hook attachment area.

### Bio-Medical Devices Intl

17171 Daimler Street  
Irvine, CA 92614  
www.maxair-systems.com  
800-443-3842