




CAUTION

Before use, thoroughly read MAXAIR® CAPR® P/N 03521015 User Instruction (received with all CAPR Helmets and available at www.maxair-systems.com, User Instructions).

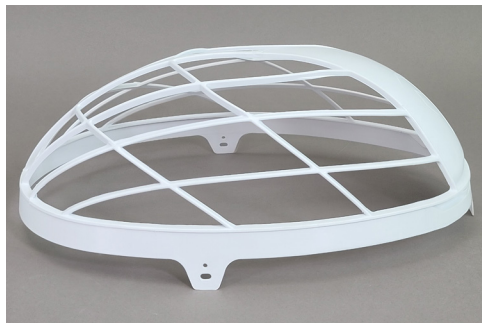

REF 2051-08
(3/box)

P/N 03531406

Intended Use

The SnapOn Cage serves two uses. During shipping, it protects the MAXAIR Helmet Blower Assembly, Motor and Fan. During use, it provides for air flow channels for MAXAIR Hoods and for Cuffs and Shrouds when used with OTHER THAN 216X-XX cartridge style filters.

The 2051-08 is included as a component within the MAXAIR 2087-03 Helmet Assembly:



WARNING

The SnapOn Cage is not to be used for any MAXAIR Systems configurations that require use of a MAXAIR Filter Cartridge style filter.

Symbol Definitions



Warning, Caution, or Note

REF

Catalog Number

P/N

Part Number



WARNING

Failure to follow User Instructions P/N 03521015 and the instructions contained herein may be hazardous to the user's health.

Use only if package is received unopened and contents are undamaged. If damage is noted, contact the shipper for replacement or repair.

Prior to using any MAXAIR® System or component, be sure to be familiar with the system's NIOSH approved configuration.

DO NOT use if any component is damaged. If any components are damaged or contaminated and therefore unfit for safe and effective use, they should be replaced immediately.

Only trained and experienced personnel who have read and understand the User Instructions should use MAXAIR Products.

The institution using this product in any application is responsible for determining the appropriateness of this equipment relative to regulatory requirements. Bio-Medical Devices Intl, Inc. does not recommend the appropriate systems for a particular institution or facility.

Use only MAXAIR Systems/ NIOSH approved compatible components.

NOT for use in atmospheres immediately dangerous to life or health (IDLH), and atmospheres containing less than 19.5% oxygen, or more than 25% oxygen.

Flammability Level I: fabric may burn if exposed to open flame.

Follow current local regulations governing biohazard waste to safely dispose of single use MAXAIR Products.

If you need more information, contact your BMDI Sales Representative, or call BMDI customer service at 1-800-443-3842.

Specifications

Material

Nylon

Recommended System Temperature Limits

Use/Handling: 0°C to 54°C at a maximum 80% Relative Humidity.

Charging: 0°C to 45°C at a maximum 80% Relative Humidity.

Storage: 0°C to 35°C at a maximum 80% Relative Humidity.

Regulatory

NIOSH

Disassembly - 2051-08 Cage From Helmet



1. Unsnap one Cage side Snap Tab from its Helmet side Snap.



2. Unsnap the other Cage side Snap Tab from the Helmet other side Snap.



3. Lift the Cage up and back off the rear Helmet Snap, and off the Helmet.

Assembly - 2051-08 Cage To Helmet



1. Place the 2051-08 Cage over the top of the Helmet.



2. Snap the Cage rear Snap Tab to the Helmet rear top Snap.



3. Snap one Cage side Snap Tab its respective Helmet side Snap.



4. Snap the other Cage side Snap Tab to the Helmet other side Snap.