

**CAUTION**  
CAPR Base System User Instructions, P/N 03521015, must be thoroughly read and understood before using any MAXAIR CAPR System. (Included with each shipped Helmet and at maxair-systems.com, Resources, User Instructions.)

**REF** 2099-26  
10 Sets of 2 Hangers

**REF** 2099-28  
6 Sets of 2 Hangers

**P/N** 03731071



### Intended Use

The MAXAIR 2099-26 (10 sets or 2 each) and 2099-28 (6 sets or 2 each) Hanger Kits provide Hangers as needed on MAXAIR Helmets when used with Gaiter and Bonnet Headcovers.

Hangers are not to be used with other Headcovers.

**CAUTION**  
Hangers are only to be used with MAXAIR Gaiter and Bonnet Headcovers.

### Symbol Definitions

**Warning, Caution, or Note**

**REF** Catalog Number **P/N** NIOSH Number

**WARNING**

Use only if package is received unopened and contents are undamaged. If damage is noted, contact the shipper for replacement or repair.

DO NOT use if any component is damaged. If any components are damaged or contaminated and therefore unfit for safe and effective use, they should be replaced immediately.

Failure to follow User Instructions P/N 03521015 and the instructions contained herein may be hazardous to the user's health.

Only trained and experienced personnel who have read and understand the User Instructions should use MAXAIR Products.

Prior to using any MAXAIR® System or component, be sure to be familiar with the system's NIOSH approved configuration.

The institution using this product in any application is responsible for determining the appropriateness of this equipment relative to regulatory requirements. Bio-Medical Devices Intl, Inc. does not recommend the appropriate systems for a particular institution or facility.

Use only MAXAIR NIOSH approved respirator configurations.


NOT for use in atmospheres immediately dangerous to life or health (IDLH), and atmospheres containing less than 19.5% oxygen or more than 25% oxygen

Flammability Level I: fabric may burn if exposed to open flame.

Follow current local regulations governing biohazard waste to safely dispose of single use MAXAIR Products.

If you need more information, contact your BMDI Sales Representative, or call BMDI customer service at 1-800-443-3842.

### Hangers Assembled to MAXAIR Helmet

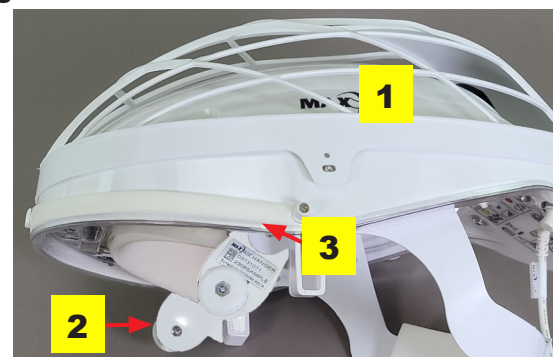

1. MAXAIR Helmet	2. Hanger on right side
	3. Hanger on left side

### Specifications

#### Materials:

Lexan  
Polycarbonate

#### MAXAIR Recommended System Temperature Limits:

Use/Handling: 0°C to 54°C at a maximum 80% Relative Humidity.  
Charging: 0°C to 45°C at a maximum 80% Relative Humidity.  
Storage: 0°C to 35°C at a maximum 80% Relative Humidity.


All MAXAIR Systems, components, and disposables are latex free.

Individual  
Hanger




Hanger  
Button

### Connect Hangers to Helmet

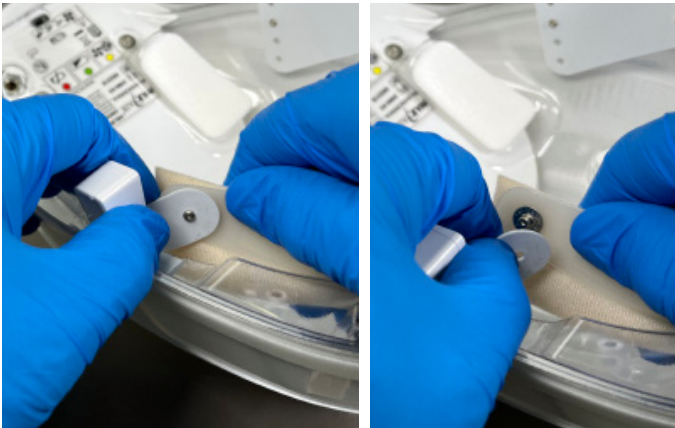

1. Unsnap one Headband Side Strap from the Headband Front Strap.

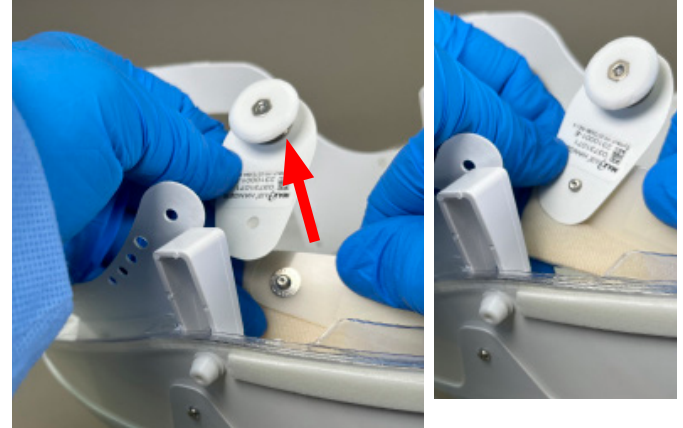

2. Place one Hanger attachment hole over the snap on the Front Strap and snap them together securely.



3. Place the Side Strap over the Hanger and snap it to the Front Strap to secure the three together.



#### CAUTION


Ensure that the Hanger Button (red arrow) is facing outward, away from the wearer's face, on both sides.

4. Repeat steps 1-3 for the other side of the Helmet.

### Disconnect Hangers from the Helmet



1. For one side Hanger, unsnap the Side Strap off that Hanger and remove the Hanger off the Front Strap.

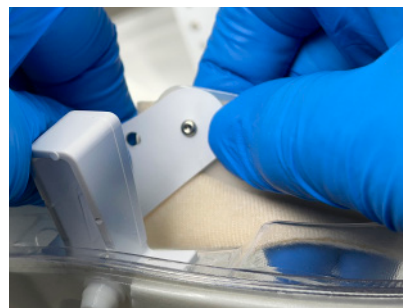

2. Re-snap the Side Strap to the Front Strap.

3. Repeat 1-2 for the other side Hanger.