

CAUTION
Before use, refer to the CAPR® User's Instructions, P/N 03521015 (received with all CAPR Helmets) for details.




REF 2400-208
P/N 07831343

Intended Use

The MAXAIR™ 2400-208 Peel Away Lens is for use with MAXAIR System Face and Head Covers with Integrated Lenses.

The Peel Away provides three layers of removable thin film lenses to protect the Head/Face Cover's integrated Lens.

Example Configuration DLC Lens Cuff with Peel Away

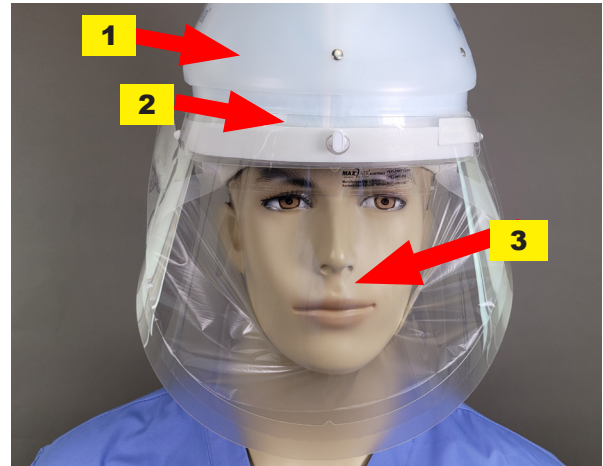

1. CAPR Helmet with FCC	2. DLC Lens Cuff
3. 2400-208 Peel Away Lens	4. 2400-208, Package of 10

Warning, Caution, or Note
REF Catalog Number **P/N** NIOSH Number

WARNING

Use only if package is received unopened and contents are undamaged. If damage is noted, contact the shipper for replacement or repair.

Prior to using any MAXAIR™ System or component, be sure to be familiar with the system's NIOSH approved configuration.

DO NOT use if any component is damaged. If any components are damaged or contaminated and therefore unfit for safe and effective use, they should be replaced immediately.

Only trained and experienced personnel who have read and understand the User's Instructions should use this product.

Failure to follow the User's Instructions, these and in the MAXAIR User's Instructions, P/N 03521015, may be hazardous to the user's health.

The institution using this product in any application is responsible for determining the appropriateness of this equipment relative to regulatory requirements. Syntech Intl. does not recommend the appropriate systems for a particular institution or facility.

Use only MAXAIR NIOSH approved compatible components.

NOT for use in atmospheres immediately dangerous to life or health (IDLH), and atmospheres containing less than 19.5% oxygen, or more than 25% oxygen.

Flammability Level I: fabric may burn if exposed to open flame.

Follow current local regulations governing biohazard waste to safely dispose of used shrouds.

If you need more information, contact your MAXAIR Representative, or call MAXAIR customer service at 1-800-443-3842.

Recommended System Temperature Limits	
Use/ Handling	0°C to 54°C at a maximum 80% Relative Humidity
Charging	0°C to 45°C at a maximum 80% Relative Humidity
Storage	0°C to 35°C at a maximum 80% Relative Humidity

Regulatory


NIOSH PAPR100-N




CAUTION

Before use, always refer to the CAPR® User's Instructions, P/N 03521015 (received with all CAPR Helmets) for details.

Components; Overview of Layers and Sequence of Removal

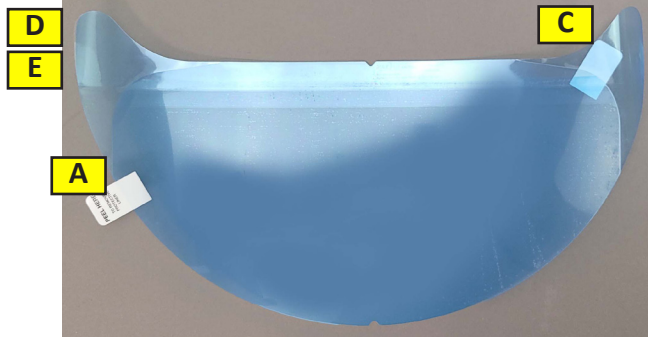

1. View of the front side of the Peel Away Lens.

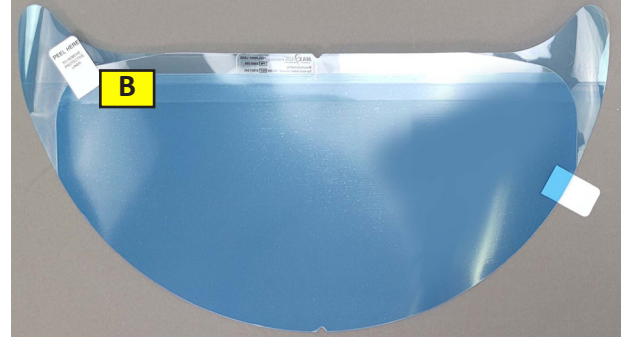

2. View from the back side of the Peel Away Lens.

**Layers and Sequence of Removing Layers
(Orientation of left and right are that of the Wearer)**

Before Assembly to the Face, Head, or Body Cover Lens

- A: Front Lens Protective Layer (Translucent Blue) Peel Tab: Remove before use. Pull tab up, to right, and off. Discard.
- B: Back Lens Protective Layer (Translucent Blue) Peel Tab: Pull tab up and pull down and off before assembling to Face/Head Cover Lens. Discard.

After Assembly to the Face, Head, or Body Cover Lens

- C: Left Layer Tab for Top Layer Peel Away Lens: Pull forward, across to the Right, and completely off. Discard.
- D: Right Layer Tab for Top Layer Peel Away Lens: Although the Top Layer has Tabs on both sides, it is recommended to use C to remove Top Layer Peel Away Lens.
- E: Right Layer Tab for Middle Layer Peel Away Lens (Behind D): Pull forward, across to the Left, and completely off. Discard.




NOTE

The Peel Away consists of three removable Lenses.

The top removable Peel Away Lens has a Layer Tab extension on both sides. It is recommended to use the Layer Tab on the Left (wearer) side (C) for removal.

The bottom removable Peel Away Lens does not have a Layer Tab extension for removal.

Preparation for Assembly

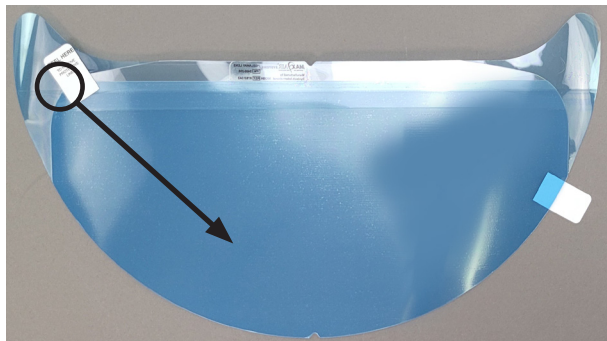

NOTE

Left and Right references relate to that of the user when the Helmet is on their head.

1. With the Peel Away Lens facing away from you, use the top-left Peel Tab to peel down the top edge of the translucent blue Back Protective Lens Cover. Pull it all the way down and across to the other (right) side, and off the Peel Away Lens. Then discard the Back Protective Lens Cover.

CAUTION
Before use, refer to the CAPR® User's Instructions, P/N 03521015 (received with all CAPR Helmets) for details.

Assemble to the Integrated Lens of the Face, Head, or Body Cover




1. Align the notch in the center top edge of the Peel Away vertically with the Helmet center mounting post position.



2. Be sure to hold the Peel Away level horizontally with its top edge parallel and very near the top edge of the Integrated Lens.



3. Press the Peel Away against the Integrated Lens all the way out to its side edges. Slide your grasp of the Peel Away against the integrated Lens and back along each side to firmly secure the entire Peel Away Lens against the Integrated Lens.

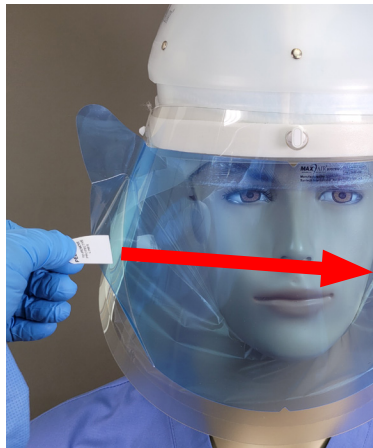

4. Grasp the Front Peel Away Lens Protector Tab, pull it away from and across and to the left until it is free from the Peel Away. Discard. The Peel Away is now assembled and ready for use.






CAUTION

Before use, refer to the CAPR® User's Instructions, P/N 03521015 (received with all CAPR Helmets) for details.

Remove Peel Away Layers as Needed



1. When it is deemed necessary to remove a Peel Away layer, grasp the Top Layer Tab on the upper left side and pull downward and to the right to remove the Top Layer Lens. Discard the Peel Away Layer as contaminated waste.

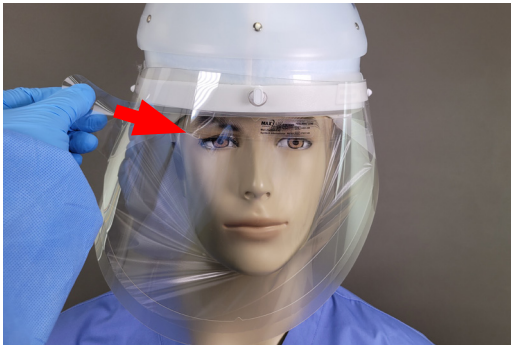

2. When it is deemed necessary to remove another Peel Away layer, grasp the Middle Layer Tab on the upper right side and pull downward and to the left to remove the Bottom Layer Lens. Discard the Peel Away layer as contaminated waste.



3. When it is deemed necessary to remove another Peel Away layer, place your index or middle fingers between the Bottom Peel Away layer and the integrated Lens and begin lifting the Bottom Peel Away Layer from the integrated Lens. Discard as contaminated waste. Continue lifting the Peel Away up and completely off the integrated Lens. Discard the Peel Away as contaminated waste.